

**Robotics**  
Anything is Possible!

## HARRTS V-5 PEN IMPLANTER



### **HARRTS V-5 Implanter Pen**

The Latest Revolution in Direct Hair Implantation ( D H I ) Technique The Most Error Free Technique of Performing Follicular Unit Implantation..

### **Salient Features of HARRTS V-5 Implanter Pen:**

- Fastest Direct Hair Implantation (DHI)
- No need to create Channels
- Very Dense Packing of Grafts
- Minimal Popping of Grafts
- Cheaper than CHOI Pen
- Per needle cost just \$ 2.5 USD as opposed to \$ 20 of CHOI Needles.
- Available in Different Sizes
- Complete Light Wt. Metal Body
- Autoclavable And Re-Usable
- Negligible Cost per Surgery



## HARRTS V-5 IMPLANTER PENS

### HARRTS V-5 Implanter Pen Needles


The Specially Engineered Needles give you a cost-effective, precision oriented and speedy solution for Implantation of the Follicular Units. Not only do these needles come in different sizes to fit individual size of grafts, but , as opposed to Choi Implanters, You also get a precise depth control

### Salient Features of HARRTS V-5 Implanter Pen:


- No need to create channels
- Very Dense Packing of Grafts per Square Centimeter
- Minimal Popping of Grafts
- Cheapest Solution, Cheaper than Choi Implanter Needles
- Per needle cost of Just \$ 2.5 USD as opposed to CHOI Needles which cost \$ 15-20 USD
- Easiest to use
- No Learning Curve



# HARRTS V-5 Implanter Pen Assembly

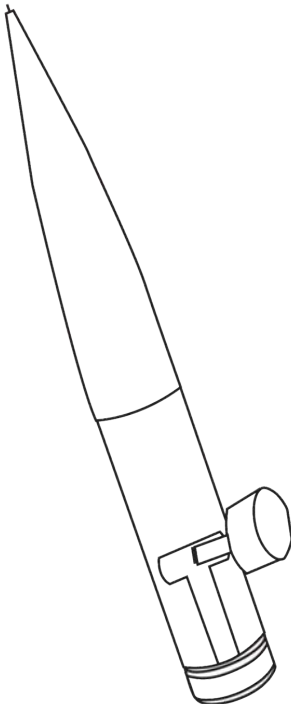

**V5 Implanter**

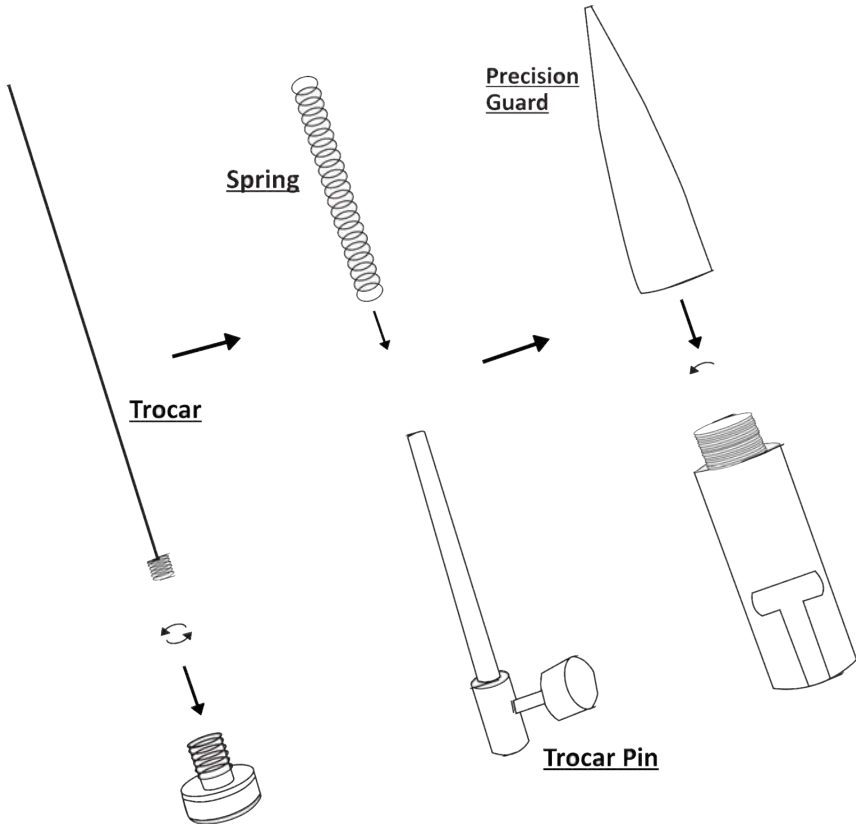

# HARRTS V-5 Implanter Pen Assembly

## Some Useful Information about the V-5 Implanter Pen

- Follow the steps shown in the above diagram to assemble and dis-assemble the implanter while loading new springs and needles in them.
- The HARRTS V-5 Pen Implanter is Autoclavable and can be reused after autoclaving it.
- The Needles are sterile and be used directly from the pack.
- The needles are disposable and should not be reused on different patients.
- The Needles come in 2 sizes, 18 gauge for 3,4 hair follicular units, and 19 gauge for 1 and 2 hair follicular units.
- At-least 5 Implanter pens should be used while performing follicular implantation case.
- About 10 Needles will be needed to complete a case of around 1500 grafts. ( 2 per pen implanter)
- The Springs are disposable and should be changed for each patient. Do not use the same spring for more than one case.
- If the same spring is used for more than one case, the implanter may not function properly.
- For any questions, email us on [info@ibrainrobotics.net](mailto:info@ibrainrobotics.net)





# Robotics

Anything is Possible!

## CONTACT DETAILS

Get in touch with us! We would like to hear from you.

### USA ADDRESS

77380 W. Sand Lake Road,  
Suite 500,  
Orlando, Florida,  
32819,  
USA

### PHONE

USA Landline | +1 352 804 6054

### WEB

[www.ibrainrobotics.net](http://www.ibrainrobotics.net)

[info@ibrainrobotics.net](mailto:info@ibrainrobotics.net)

